



Using Sensibles is as easy as 1-2-3:

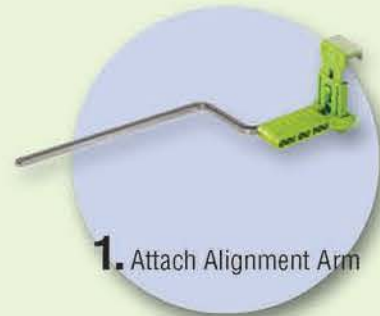

1. Attach Alignment Arm



2. Add Aiming Ring

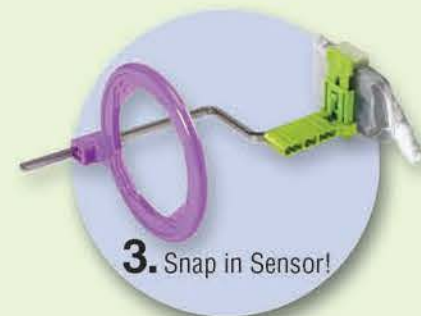

3. Snap in Sensor!



For Bite-Wings,
adjust vertical position

SENSIBLES: INSTRUCTIONS FOR USE

Assemble the appropriate instrument using the pictures on the previous pages as a guide. Aiming Rings are color-coded: purple for the posterior alignment arm, and blue for the anterior/bite-wing alignment arm. Selecting a Sensible bite block: Large (two-tone) opens wider and holds most size 1 and 2 sensors vertically; Medium holds most size 0, 1, and 2 sensors horizontally. Be sure the bite block is centered to the ring, to ensure centering to the x-ray beam.

1. Attach an Anterior, Posterior or Bite-Wing Alignment Arm to the bite block.

2. Add the corresponding Aiming Ring.

3. Raise the upper jaw of the Sensible by pressing the button on the back and pulling up. Insert your sensor. Once the sensor is positioned correctly, press down on the upper jaw to achieve a tight, locking fit. You will hear clicks as the upper jaw moves into place and locks tight.

When using the bite-wing bite block, be sure the alignment arm is centered on the sensor.

Place the Sensible Bite Block into the patient's mouth, as far back in the mid-tongue area as possible. Center the sensor to the teeth to be x-rayed. Refer to the diagrams on the previous page for placement. Instruct the patient to bite down in order to hold the instrument in place. A cotton roll can be placed between the Sensible and the opposing teeth to ensure the bite block does not shift.

4. Position the patient by moving the chair or the patient's head, allowing you to comfortably bring the x-ray tube up to the aiming ring. The alignment arm will be parallel to the x-ray tube.

5. Expose the sensor. For additional exposures in various areas inside the mouth, you may need to change aiming rings and/or alignment arms, as well as alter the sensor from a horizontal to a vertical position or vice-versa.

FD122712r1

SENSIBLES KITS AND PARTS

Sensibles Deluxe Kit with Bite-Wing 40915

6 Large (two-tone) Sensibles Bite Blocks
10 Medium Sensibles Bite Blocks
2 Anterior/Bite-Wing Aiming Rings
2 Posterior Aiming Rings
2 Anterior Alignment Arms
2 Posterior Alignment Arms
2 Bite-Wing Alignment Arms
50 Deluxe Comfee Samples

Sensibles Econo Starter Kit 40901

2 Large (two-tone) Sensibles Bite Blocks
2 Medium Sensibles Bite Blocks
1 Anterior Aiming Rings
1 Anterior Arms
1 Posterior Aiming Rings
1 Posterior Arms
25 Deluxe Comfee Samples

Sensibles Bite-Wing Kit 40912

2 Large Sensibles Bite-Wing Bite Blocks
2 Medium Sensibles Bite-Wing Bite Blocks
2 Bite-Wing Alignment Arms
2 Aiming Rings

Sensibles Large Bite Blocks 40910

12 Large (two-tone) Sensibles Bite Blocks

Sensibles Medium Bite Blocks 40909

12 Medium Sensibles Bite Blocks

~~Sensibles Large Bite-Wing Bite Blocks 40913~~

~~4 Large Sensibles Bite-Wing Bite Blocks~~

~~Sensibles Medium Bite-Wing Bite Blocks 40914~~

~~4 Medium Sensibles Bite-Wing Bite Blocks~~

Anterior/Bite-Wing Aiming Ring 408590

Posterior Aiming Ring 408600

Anterior Alignment Arm 408570

Posterior Alignment Arm 408580

Bite-Wing Alignment Arm 409150

CLEANING AND STERILIZING OPTIONS

ALL OF THE ENCLOSED INSTRUMENTS ARE AUTOCLAVABLE. Autoclave before first use.

1. **Soak** for one hour in glutaraldehyde. DO NOT USE PHENOL-BASED GLUTARALDEHYDE. Rinse and dry.

2. **Clean ultrasonically**, use a general purpose ultrasonic cleaner or detergent/soap and water. DO NOT USE SPECIAL APPLICATION CLEANERS. Rise and dry.

3. **Steam Autoclave** (distilled water only). DO NOT CHEMICLAVE OR DRY HEAT STERILIZE. DO NOT USE THE AUTOCLAVE DRYING CYCLE.

OR Cold Sterilize. CHEMICALS MAY CAUSE SKIN IRRITATION IN SOME PATIENTS.

Sensibles™

Universal Sensor Holder



- Everything you need to capture the perfect image is inside: Bite Blocks, Aiming Rings, Alignment Arms and more!
- Perfect for Anterior, Posterior and Bite-Wing images
- Adjusts easily; holds securely
- Autoclavable and reusable
- Durable and 100% latex-free


SENSIBLES DELUXE KIT CONTAINS:

6 Large (two-tone) Bite Blocks
10 Medium Bite Blocks
2 Anterior/Bite-Wing Aiming Rings
2 Posterior Aiming Rings
2 Anterior Alignment Arms
2 Posterior Alignment Arms
50 Deluxe Comfee Samples
NEW! 2 Bite-Wing Alignment Arms

*Patent Pending

Sensibles Anterior Positioning Guide



Maxillary

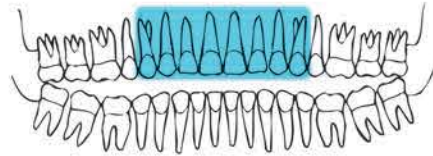


Use either Bite Block, depending on size of sensor.



Use Alignment Arm "ANTE 40857-10."

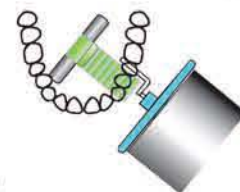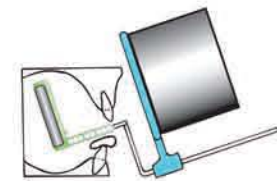


Mandibular

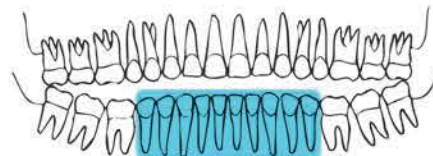

Use Alignment Arm "ANTE 40857-10."




Use either Bite Block, depending on size of sensor.

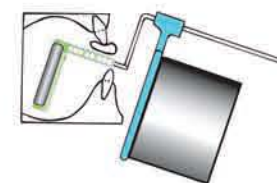

Sensibles Posterior Positioning Guide




Maxillary

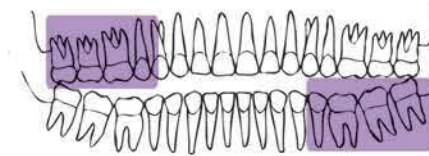


Use either Bite Block, depending on size of sensor.



Use Alignment Arm "POST 40858-10."

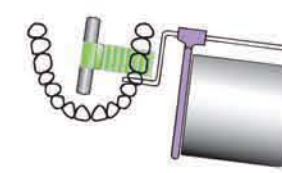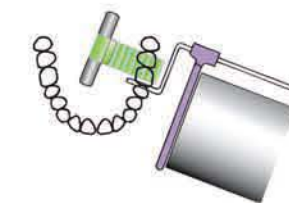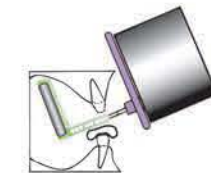



Mandibular

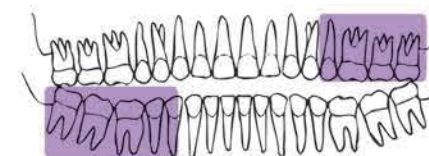



Use either Bite Block, depending on size of sensor.



Use Alignment Arm "POST 40858-10."

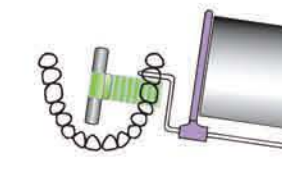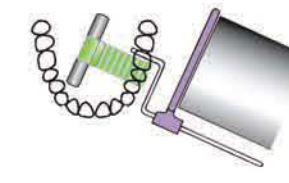


Sensibles Bite-Wing Positioning Guide

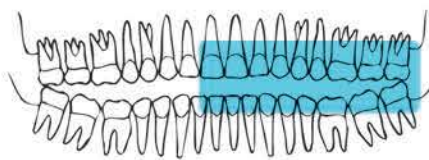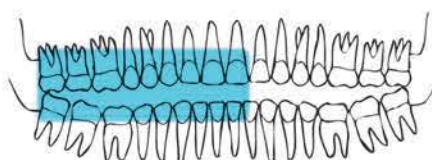

Use either Bite Block, depending on size of sensor, then simply slide Bite Block to mid-line of sensor.



Use Alignment Arm "BITE 40915-10."

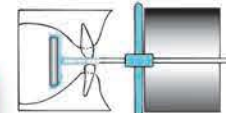

Horizontal Imaging



The Medium Sensible holds most size 2, 1 and 0 sensors horizontally.

Vertical Imaging



The Large (two-tone) Sensible holds most size 2 and 1 sensors vertically.

Bite-Wing Imaging



For Bite Wing imaging, simply slide Bite Block to mid-line of sensor.

To Insert Sensor



1. Place sensor in holder



2. Push top down

To Remove Sensor



Push button on back

We Recommend



Comfee Deluxe Sensor Sleeves
Large: 80210 Medium: 80200